


VACUETTE® 3.5ml Coagulation tubes

FILLING

Ensure that the blood fill level is within the green region.


The full height of the arrow is acceptable for coagulation testing with the correct ratio of blood to additive.

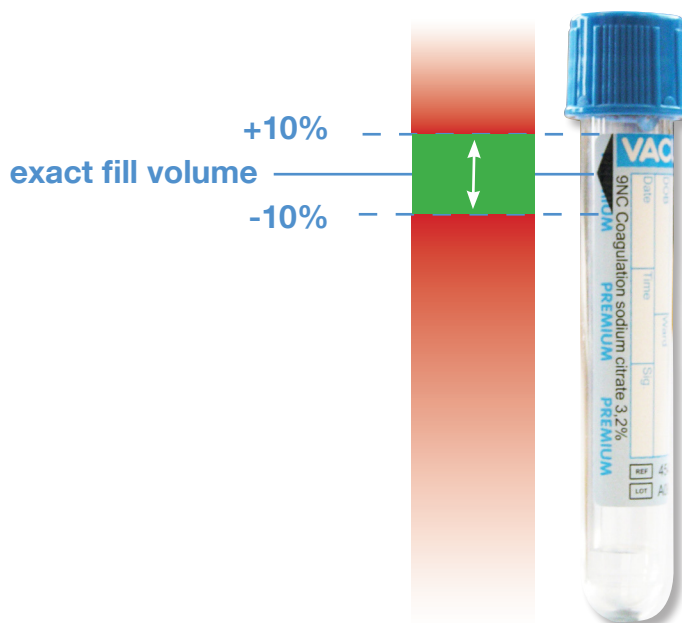

BUTTERFLIES


When using a butterfly to collect a citrate tube the air must be removed from the line by using and discarding a separate citrate tube.

Discard Tube
454327
KFK225

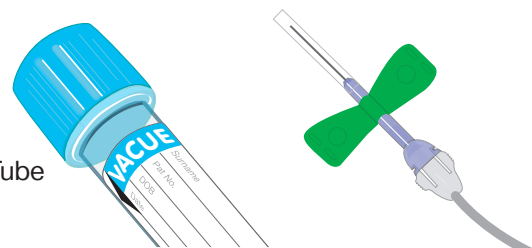

MIXING

Samples should be fully inverted 6 - 8 times.

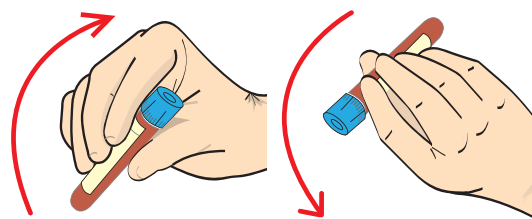

STOCK ROTATION

For best results in sample collection, ensure good stock rotation is maintained. Use tubes with oldest date first!

2019-06



followed by

2019-09



STORAGE

Coagulation Tubes should be stored at between 4 - 25°C, avoiding storage in direct exposure to the sun, near heating devices or other warm areas.

